

Telemedicine.com

General Examination Wand Setup



Step 1

Take USB cable and line it up to the back of the wand



Step 2

Gently push in the cable until it seats flush with the wand




Step 3

Plug the other end of the USB cable into a computer



Step 4

Within your software select Telemedicine.com Wand as the video camera to project the wand video image

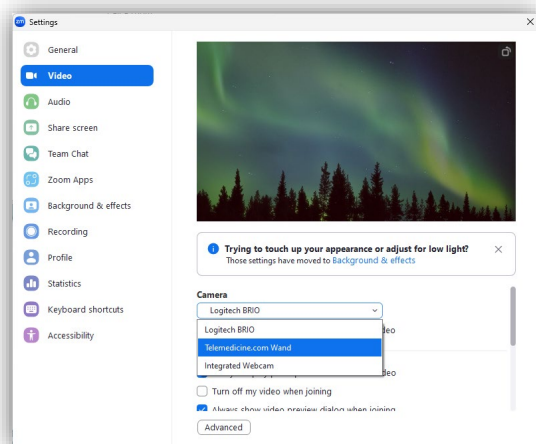

How To Apply The Sheath On The Telemedicine.com General Examination Wand

1: Hold the open end of the sheath with the paper side facing down and insert the wand lens-side-down under the white tab facing the paper.



2: Gently slide the wand until it reaches the end of the cover. Do not use it if the sheath becomes torn or punctured.



3: Peel back the plastic film top cover and discard.



4: Peel back the opposite-side paper cover and discard.



5: Gently continue pushing the wand into the cover so that the plastic film is smooth and wrinkle-free against the wand camera lens. Use as normal.

